Gebruiksaanwijzing Operating Instructions Instrucciones de uso Manuel d'instructions Istruzioni per l'uso

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6. OPERATION

Check that batteries or rechargeable batteries are in good condition. Replace flat, discharged or damaged power sources in case of doubt (observe correct polarity). Check that the lamp is tightly seated in the socket, tighten if necessary. Screw battery container or rechargeable battery insert into the handle sleeve. Insert laryngoscope blade and test operation.

7. REPLACEMENT OF EXPENDABLE PARTS

When purchasing replacements for expendable parts, such as lamps, rechargeable batteries etc., use only original parts. Installing unauthorized parts can lead to malfunctioning, for which we are not liable. This will also void the guarantee.

7.1 Lamps

Unscrew the battery container or rechargeable battery insert from the handle sleeve. Take care that the new lamp has the correct voltage and the insulation collar on the threaded base of the new lamp is present. Screw in new lamp. Insert laryngoscope blade and test operation.

Important: allow lamp to cool down before replacing it. Do not touch the lamp with your fingers or rubber gloves because the light output may be reduced.

7.2 Batteries

Unscrew battery container from the handle sleeve. Always replace both batteries. Remove batteries, install new batteries and observe correct polarity. Screw battery container into the handle sleeve. Insert laryngoscope blade and test operation. Dispose of the used batteries in accordance with environmental regulations.

7.3 Rechargeable battery insert (NiCd)/LED rechargeable battery insert (Li-Ion)

Replace handle with rechargeable battery insert or rechargeable battery insert separately with a new fully charged unit. Charge handle with rechargeable battery insert or rechargeable battery insert separately with an inductance charger. (Art. no. 45.49.98 and 49.49.99).

Important: The rechargeable battery insert contains breakable ceramic parts. Avoid impacts and mechanical loading.

8. OPERATING AND STORAGE CONDITIONS

8.1 Rechargeable battery insert (NiCd)

	Temperature	rel. humidity
Storage	-20°C ... 60°C	10% ... 90%
Operation	10°C ... 40°C	30% ... 70%

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CARE INSTRUCTIONS FOR COLD LIGHT AND WARM LIGHT LARYNGOSCOPES

Table of Contents

- General instructions
- Disassembly and preparation for cleaning
- Manual cleaning and disinfection
- Machine cleaning and disinfection
- Sterilization
- Operation
- Replacement of expendable parts
- Operating and storage conditions
- Safety information
- Disposal

1. GENERAL INSTRUCTIONS

This instrument is not supplied in a sterile condition and therefore must be cleaned, disinfected and if necessary sterilized before being used for the first time and before every subsequent use.

Laryngoscopes are high-quality instruments that must be treated with care. Follow the instructions below. They are designed to ensure that your instrument operates correctly and reliably.

The fit component on the handle and laryngoscope blade is compatible with ISO 7376-3 and can therefore be used with instruments from other manufacturers that conform with ISO 7376-3.

MEDICON eG as the supplier of these products is not liable for immediate or consequential damages caused by or arising from improper use or handling, especially through non-observance of the enclosed operating instructions or through improper cleaning and maintenance.

2. DISASSEMBLY AND PREPARATION FOR CLEANING

Important: hot lamp! Allow it to cool down before touching it.

If necessary, remove the laryngoscope blade from the handle. Unscrew the battery container or rechargeable battery from the handle for cleaning and sterilization. Laryngoscopes with removable light conductors must be dismantled, because cleaning and sterilization must be done when they are disassembled.

The battery insert, batteries and rechargeable batteries must never be cleaned with water or aqueous solution. Wipe these parts with a cloth moistened with alcohol.

Processing of used instruments should be started as soon as possible, since incrustations may cause irreparable damage. Remove heavy soiling with a moist (preferably alcohol-free) disposable cloth or a soft brush. Rinse thoroughly with water.

3. MANUAL CLEANING AND DISINFECTION

Important: never clean batteries, battery container and rechargeable batteries with water or aqueous solution; protect contacts from moisture.

Completely immerse and soak the laryngoscope blade and handle sleeve in cleaning/disinfectant solution. Afterwards thoroughly rinse with demineralised water and dry with sterile compressed air. Any chemical may be used that is approved for these instruments according to the manufacturer's information. The manufacturer's instructions regarding the mixing ratio and immersion time must be observed precisely. The following solutions have been examined with regard to the compatibility of materials: Gigasept, Ivisol, Korsolin, Orbiphen, Bacillotox, Baktol, Cidex. The solutions must be replaced regularly since the risk of corrosion increases with standing time and contamination of the solution.

The maximum immersion time of 60 minutes must not be exceeded. After cleaning rinse thoroughly with demineralised water. Clean the lens of the Li-ion battery with a cotton bud soaked in 70% isopropyl alcohol. Wipe the battery container, batteries and rechargeable battery insert with a disposable cloth soaked in disinfectant and allow to dry.

Important: contact with other cleaning agents, disinfectants or other chemicals can lead to irreparable damage. Instruments must never be immersed in physiological saline solution since prolonged contact leads to pitting. Follow the procedures and products approved by the instrument manufacturer for cleaning, sterilization and care of the instrument.

4. MACHINE CLEANING AND DISINFECTION

Note: Machine cleaning and disinfection are not permitted for warm light laryngoscope blades.

Consult the manufacturer of the cleaning machine for the selection of cleaning programs. Usually there are special programs for endoscopic instruments (possibly even for telescopes) which are more suitable for cleaning and protecting materials than general programs. Clean the laryngoscope blade and handle sleeves in cleaning and disinfection machines at up to 93 °C.

5. STERILIZATION

Laryngoscopes must be clean and dry before steam sterilization. Sterile compressed air is most suitable for drying. Laryngoscopes can be absolutely sterilized at 121° C (=250° F) for 20 min. at 2 bar, or at 134° C (=273° F) for 5 min. at 3 bar. Make certain that the laryngoscopes are allowed to cool down slowly.

Gas sterilization (formaldehyde, ethylene oxide), plasma sterilization (STERRAD®) and chemical sterilization (e.g. Steris®) is also possible. The methods described have been validated.

Important: hot air sterilization is not permitted for warm light laryngoscopes.

Please note that successful sterilization is only possible if an acknowledged and validated process is followed.

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