Induktives Ladegerät
Kat.-Nr. 45.49.98

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Inductive Charger
Cat. No. 45.49.98

Contents

- General
- Safety information
- Operation
- Recommendation
- Cleaning the charger
- Power pack

General

The inductive charger is used for contactless charging of the MEDICON rechargeable battery unit: Cat. No. 45.48.26 Lithium-ion rechargeable battery with LED

Standard conformity: EN60601, Operation outside the patient environment.

Safety information

Explosion hazard! Handpiece and inductive charger are not suitable for operation in explosion-hazard areas.

Never store magnetic data storage media (floppy discs, magnetic tape, credit/check cards etc.) directly next to or on top of the inductive charger, as this could corrupt the data.

Operation

The supply voltage stated on the nameplate must be identical to the existing supply voltage. Insert the power plug into the socket outlet. There are five LED displays. After the charging station has been connected to the power supply, all five displays light up simultaneously for approx. 1 sec. as a self-test. The green LED 1 indicates "Ready to operate". The four other LED displays indicate the charging status of the two charge bays. Two symbols in each case belong to the charge bay above them. The "orange" LED 2 indicates the "charging" condition (empty battery symbol). The "green" LED 3 indicates the "fully charged" condition (full battery symbol). The charging time for an Li-ion battery unit is approx. 14 hours. The charging time for NiCd batteries is approx. 6 hours. After this time, the charging current is switched off and current pulses are then fed out for a trickle charge in the pulse/pause ratio of 8:1. The battery units can be placed in the charging station for charging either with or without handpiece.

Recommendation

It is advantageous to store the batteries in the charger outside the unit, in order to ensure that they are constantly ready for use. In this respect, the batteries can also be removed before the end of the recharge, since in any event it is guaranteed that more current will be fed in than is consumed.

Cleaning the charger

Remove the power plug before cleaning. Clean the charging station by wipe disinfection with a surface disinfectant approved for the surfaces of medical products.

Power pack

Adapter for national plugs of the EU, UK, USA/J and Australia.
LED "green" when Power "on".
Voltage 18V +/- 10%, 840 mA.
The operating temperature for 100% performance is between 0° and 40°C.
Class II SELV: IEC 60601-1, UL 2601, VDE, CE medical class B/BF/CF

MEDICON eG, as supplier of these products, is not liable for defects caused by or arising from improper use or handling, especially through non-observance of the enclosed operating instructions or through improper cleaning or maintenance.

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