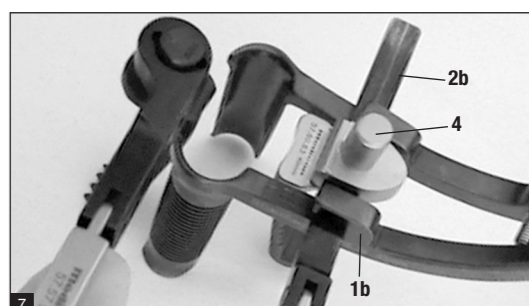
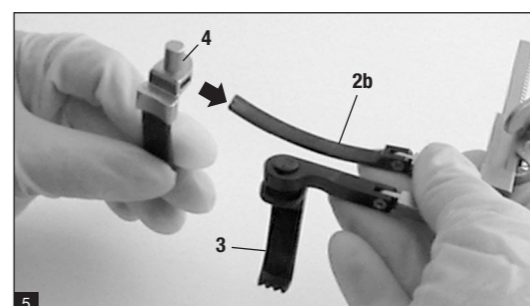
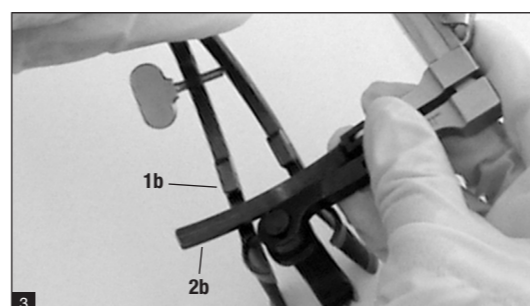
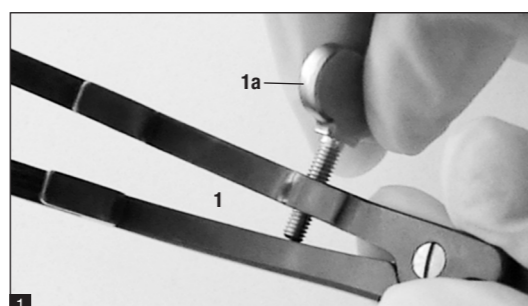
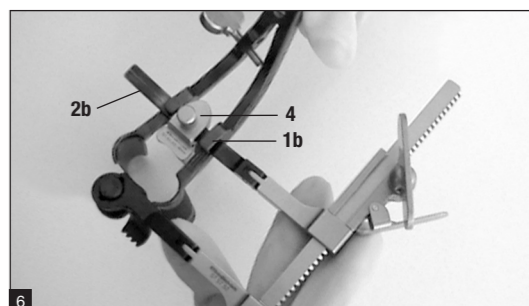
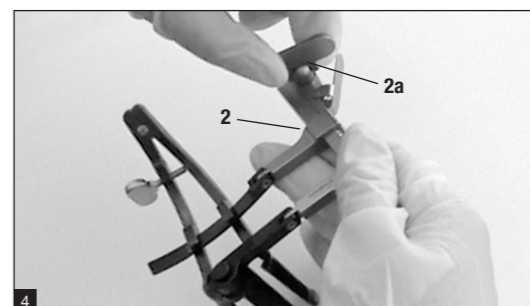
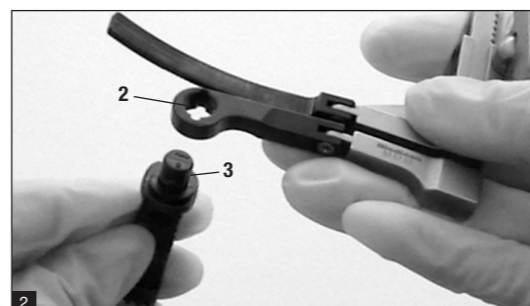
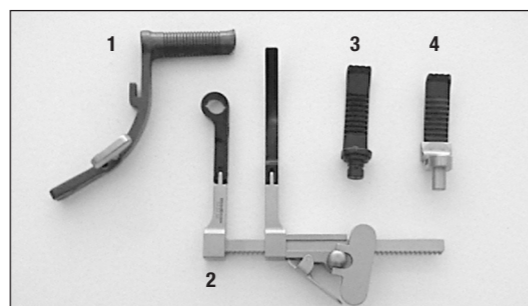
RX ONLY!

Gebrauchsanweisung
Operating Instructions
Manual de instrucciones
Mode d'emploi
Istruzioni d'uso

DE	EN	ES
Systemübersicht	System overview	Sinopsis del sistema
1 Spekulum (versch. Längen)	1 Specula (various lengths)	1 Espéculo (distintas longitudes)
2 Gegensperrerr	2 Counter-retractor	2 Contrasperrador
3 Laterale PEEK Valve (versch. Längen)	3 Lateral PEEK valve (various lengths)	3 Valva lateral PEEK (distintas longitudes)
4 Mediale PEEK Valve (versch. Längen)	4 Medial PEEK valve (various lengths)	4 Valva medial PEEK (distintas longitudes)

FR	IT
Aperçu du système	Panoramica del sistema
1 Spéculum (différentes longueurs)	1 Speculum (in diverse lunghezze)
2 Contre-sperrateur	2 Contro-divarcatore
3 Valve PEEK latérale (différentes longueurs)	3 Valva PEEK laterale (in diverse lunghezze)
4 Valve PEEK médiane (différentes longueurs)	4 Valva PEEK mediale (in diverse lunghezze)

**„PICCOLINO“-SPREIZSYSTEM**

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– DIN EN 868,
 – DIN EN ISO 1607,
 – suitable for steam sterilization (vapor permeable),
 – adequate temperature resistance up to 138°C.
 Sterilization equipment and sterilization wrapping must match both the wrap contents and the employed sterilization method.

Sterilisation
 Taking into account the respective national regulations, the following method must be employed for sterilization:
 – Vacuum autoclave with triple vacuum and adequate drying of the products (Vacuum minimal 15 Minutes)
 – Steam sterilizing complying with DIN EN 13060 or DIN EN 285 and validated in accordance with DIN EN ISO 17665-1.
 – Sterilization time and temperature: at least 5 minutes hold time at 134°C

It is essential to attain a Sterility Assurance Level of 10⁻⁴.

Storage
 Reprocessed sterile instruments must be stored in a suitable reusable sterilization container in a dry, dark, cool, and semi-sterile place, protected from dust and free from vermin. To avoid the development of condensation, major temperature fluctuations should be avoided during storage. Chemicals must be stored together with instruments.
 The walls, floors, and ceilings of the storage room should be smooth and easy to clean and disinfect. Shelves must be at least 30 cm off the floor. The duration of storage permitted depends on the type of sterile barrier system employed and the storage conditions. This storage period must be established by the operating authority.

Further information on reprocessing
 Validated machine cleaning and disinfection is always preferred over manual cleaning because of the greater certainty of the method. Good cleaning helps to preserve value and is a precondition of successful sterilization. During machine processing, the following points should be noted:
 – Correct loading of the trays for rinsing is a precondition for effective machine processing. Trays must not be overloaded.
 – Rinsing shadows due to large instruments must be avoided.
 – The instruments must be placed or stored based on their susceptibility to mechanical damage in order to prevent them from becoming damaged.

The times and temperatures specified in these reprocessing instructions are minimum requirements and must not be less than those stated here. If they are to be reduced for technical reasons, this must be validated by the operating authority. Exceeding the stated times and temperatures is always possible but leads to increased stress on the material, which may result in premature ageing of the instruments. The use of other sterilization methods is outside our responsibility.

Information on validation of the processing
 Validation was performed with the following equipment, materials, and chemicals:

Cleaning and disinfection device:	Type Miele Desinfektor G 7735 CD and 7836 CD compartment cart for surgical instruments
Cleaning agent alkaline:	neodisher® Fa. Dr. Weigert GmbH & Co. KG

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EN

Please read these instructions carefully before clinical application and keep them safe and close to hand.

Table of contents
 1. General notes
 2. Purpose and proper use
 3. Indications
 4. Contraindications
 5. Possible side effects and complications
 6. CT & MRI instruction
 7. Handling
 8. Decontamination, Cleaning and sterilization
 9. Liability
 10. Signs and symbols

1. GENERAL NOTES
 The „piccolino“-spreading system is supplied unsterile and must be disinfected, cleaned and sterilized prior to use. Please observe the following instructions. This will guarantee that the instrument functions faultlessly and reliably.

2. PURPOSE AND PROPER USE
 The „piccolino“-spreading system is used to spread and maintain surgical access in lumbar and cervical spine surgery. The „piccolino“-spreading system is for miniaturised subperitoneal paravertebral access techniques, as well as for transmuscular ones.

3. INDICATIONS
 – Micro-discectomy
 – Laminectomy
 – Foraminotomy
 – Spinal canal stenosis
 – Lumbar fusion
 – Posterior lumbar intervertebral fusion (PLIF)
 – Transforaminal lumbar interbody fusion (TLIF)

4. CONTRAINDICATIONS
 Applications contrary to the intended use of the product.

5. POSSIBLE SIDE EFFECTS AND COMPLICATIONS
 With excessive loads due to pulling and pushing forces during the spreading action, irritation, local circulatory disorders in the skin and damage to the soft tissue and muscles may occur.
 This may cause tissue necrosis, impaired wound healing and infections. Furthermore, an overloading of the spreading system may lead to breakage of the instrument, or parts of it.

Warning notes and precautionary measures
 So that there is no overloading of the spreading system or tensions being applied to soft tissues, it is the responsibility of the surgeon to extend the surgical wound if necessary, in order to bring about a corresponding degree of relief.

6. CT AND MR INSTRUCTION
 The „piccolino“-spreading system is not suitable for use with MRI (magnetic resonance imaging) technology due to its material properties.

7. HANDLING

7.1 Assembly instructions
 Insert a speculum of sufficient length flush with the skin incision.
Figure 1: Spreading the speculum (1) by turning the wing screw (1a).
Figure 2: The length of the lateral valve (3) is determined by a trial insertion, and it is then fixed in the valve holder of the counter-retractor (2).

Figure 3: The toothed rack of the counter-retractor (2b) is inserted in the holder (1b) of the spread speculum.
Figure 4: Expanding the counter-retractor (2) by turning the wing screw (2a).

Figure 5: An additional medial valve may also be needed for extraforaminal access. The medial valve (4) is fixed in the toothed rack of the counter-retractor (2b). The medial valve (4) is normally 15 mm shorter than the lateral valve (3).

Figure 6-7: In the event of suspected prion contamination, incineration of the instrument is recommended according to the vCJD task force final report.
IF vCJD IS EXCLUDED:
 Continue to use after instrument processing is completed. Otherwise, the instrument, which has been contaminated or where contamination cannot be ruled out, must be disposed of as incinerated waste.

8. DEKONTAMINATION, CLEANING AND STERILIZATION
 The following vCJD prion-specific protective measure is indicated when processing instruments.

In the event of diagnosis of a definite or probable case of vCJD.
 If it is not possible to use disposable products, the instrument used, which has been contaminated or where contamination cannot be ruled out, must be disposed of as incinerated waste.

If prion contamination is suspected:
 In the event of suspected prion contamination, incineration of the instrument is recommended according to the vCJD task force final report.

IF vCJD IS EXCLUDED:
 Continue to use after instrument processing is completed. Otherwise, the instrument, which has been contaminated or where contamination cannot be ruled out, must be disposed of as incinerated waste.
In the case of a non-identifiable vCJD illness.
 Even if nothing is known about the presence of a prion disease, two processing procedures should be used with at least partial efficacy against prions – e.g. mechanical alkaline cleaning combined with steam sterilization.
 If mechanical alkaline cleaning or another cleaning procedure with proven efficacy against prions is not used and the medical devices in question are in contact with risk tissues (CNS, eyes, lymphatic tissue), the RKI recommends a prolonged sterilization time of 18 minutes at 134°C.

Instruments made of stainless steel must not be placed in physiological saline solution (NaCl solution) as prolonged contact leads to corrosion such as pitting and stress-corrosion cracking. Only cleaned and disinfected instruments may be sterilized.

Reprocessing restrictions
 Frequent reprocessing has few effects on these instruments. The end of their working life is normally determined by wear and damage through use.

Reprocessing restrictions
 Frequent reprocessing has few effects on these instruments. The end of their working life is normally determined by wear and damage through use.

use. If there are any reprocessing restrictions, this is noted in the instructions for use of the specific product.

Location of use
 – Directly after use, gross contamination must be removed from the instruments with a disposable cloth/paper.
 – Fixing agents or hot water (>40°C) must not be used as this leads to fixation of residues and can affect cleaning outcome.
 – The instruments should be conveyed promptly to processing.
 – Dry disposal preferred.

Preparation for decontamination
 Articulated instruments must be processed in open position. For rinsing, the instruments must be placed in instrument holders suitable for machine cleaning. The instrument holders (e.g. wire trays) must allow subsequent cleaning in an ultrasonic bath or in the cleaning and disinfection device (CDD) where they are not impeded by acoustic or rinsing shadows.

Pre-cleaning
 – Place the instruments in cold water for 5 minutes;
 – Brush the instruments (plastic brushes) under cold water until all visible soiling is removed;
 – Internal spaces, threads, and holes are each rinsed with the water pistol for 10 seconds and brushed again;

Cleaning: machine
 The cleaning and disinfection device (CDD) must meet DIN EN ISO 15883-1 requirements.

– Pre-rinse 1: 1 minute with cold demineralised water, without additive;
 – Emptying;
 – Pre-rinse 2: 3 minutes with cold demineralised water, without additive;
 – Emptying;
 – Cleaning: with demineralised water, heat to 55°C and wash/clean for 10 minutes, add cleaning agent at 45°C, alkaline cleaning agent, strength 0.5%;
 – Emptying;
 – Neutralization: 3 minutes with warm water (>40°C) with addition of neutralizer, strength 1 ml / l;
 – Emptying;
 – Final rinse:

Gebrauchsanweisung Operating Instructions Manual de instrucciones Mode d'emploi Istruzioni d'uso

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