Gebrauchsanweisung
Operating Instructions
Manual de instrucciones
Mode d'emploi
Istruzioni d'uso

CE 0123

Rx Only!

Medicon eG · Gänsäcker 15
 78532 Tuttlingen · Germany
 Tel.: +49 (0) 74 62 / 20 09-0
 Fax: +49 (0) 74 62 / 20 09-50
 E-Mail: sales@medicon.de
 Internet: www.medicon.de

DE

SCALP CLIP "RANEY SHARK"

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EN

SCALP CLIP "RANEY SHARK"

Please read these instructions carefully before clinical application and keep them safe and close to hand. The instructions must be followed.

MEDICON eG, as the company marketing these products, does not assume any liability for direct or subsequent damage resulting from improper use or handling, in particular as a result of failure to observe the instructions for use or through incorrect care or maintenance.

“Raney Shark” scalp clips should only be used by physicians and surgeons who have sufficient experience with operations on the skull. They must be used for their intended purpose (see intended use).

Contents

- General Information
- Purpose and proper use
- Indication
- Combination product
- Application/Handling of scalp clips
- Decontamination, cleaning and sterilization
- Storage and transport
- Liability
- Signs and symbols

1. GENERAL INFORMATION

The “Raney Shark” scalp clips are EO sterilised and supplied in sterile packaging. Please observe the following instructions. This will guarantee that the implant functions faultlessly and reliably.

2. PURPOSE AND PROPER USE

“Raney Shark” scalp clips are used for arrest of bleeding on the scalp. They are intended to be used by surgeons with sufficient ex-

perience in neurosurgery in hospitals and medical practices. They are single use products supplied in sterile packaging and should only be used together with the specified combination products.

3. INDICATION

Achieving hemostasis after an incision on the scalp in neurosurgical interventions demanding skull opening.

4. COMBINATION PRODUCT

Only Raney forceps REF 57.85.05 by Medicon should be used for the “Raney Shark” REF 57.85.03A scalp clips.

5. APPLICATION/HANDLING OF SCALP CLIPS

The scalp clips “Raney Shark” are designed and constructed for single use only on a single patient and may not be reused. The “Raney Shark” scalp clips and their packaging must be checked before use to make sure that they are intact and that the expiry date is adhered to.

Products where the packaging or the product itself is damaged or where the expiry date has already passed must not be used under any circumstances.

Defective products and products that have passed their expiry date must be immediately rejected.

Raney Shark” scalp clips must only be used with Raney forceps REF. 57.85.05 by Medicon. If they are used with any other forceps this can lead to damage of the product and a risk of injury to the patient or a third party.

Products from sterile packaging which has already been opened or damaged must not be used!

6. DECONTAMINATION, CLEANING AND STERILIZATION

The “Raney Shark” scalp clips are sterile when supplied and must not be re-sterilised!

7. STORAGE AND TRANSPORT CONDITIONS

Single use products in sterile packaging must be stored and protected in a dust-free, temperature-controlled storage area. Wide variations in temperature should be avoided during storage.

8. LIABILITY

In the event of discrepancies between the english and the german version of these instructions for use only the german version shall be applicable. The information in these instructions for use applies only to the product with which the instructions for use are supplied. The instructions for use for the current product in each case can be downloaded on the Medicon extranet at www.medicon.de/extranet or requested directly from MEDICON eG.

MEDICON eG assumes no liability for damages caused by improper use, care or maintenance or non-compliance with the restrictions for use and other guidelines in the instructions for use. The liability for defects by MEDICON eG is also not applicable in case of changes or repairs to the product without prior written consent from MEDICON eG as well as in case of repairs which were not carried out by workshops authorised by MEDICON eG or the Medicon Repair Service (MRS).

For further questions on implants or instruments, please contact MEDICON eG.

Federal law and regulations in the United States restrict the sale of this device to a physician or hospital or on prescription.

9. SIGNS AND SYMBOLS

	Follow instructions
	Order number
	CE label acc. to directive 93/42/EEC
	Sterilized with ethylene oxide
	Single use product
	Usable up to
	Warning
	Batch number
	Manufacturer
	Limit temperature
	Not resterilizable
	Keep away from sunlight
	Store in a dry place
	Limit humidity
	Do not use if packaging is damaged!
	Production date
Rx Only	Prescription use only

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