Schraubenhaltevorrichtung 68.30.06

Abb. 1

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Screw holding device 68.30.06

Fig. 1
Screw holding device,
consisting of:

68.30.08 retaining spring (1 pc.)
68.30.07 fixation clamps (2 pc.)

and 1 screwdriver for mounting purposes
(no medical device, please do not sterilize)

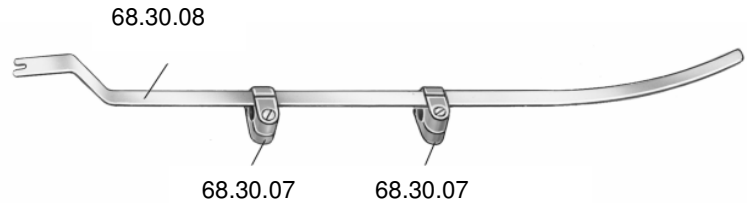
Fig. 1

Care instructions:

You have purchased a high-quality accessory for the angular screwing instrument art. no. 68.30.01, which was manufactured with meticulous care. In order to ensure that the instrument continues to function to your entire satisfaction, please comply with the instructions for use no. G683001 for the angular screwing instrument (care instructions for angular screwing instrument and care/preparation instructions for hand pieces with INTRA coupling).

Preparation, sterilization:

The screw holding device can be prepared assembled together with the angular screwdriver. When cleaning manually, remaining tissue and blood must be removed by using a soft brush (no metal brush). After preparation, carry out a cleanliness check. Stained or damaged components must be segregated and specially treated (such as thorough cleaning / repair) if necessary.

When completely new, the holding device must undergo manual or machine preparation prior to use.

Recommended method of sterilization: Steam sterilization (134 °C, 2 bar, exposure time at least 5 minutes or 121 °C, 1 bar, exposure time at least 15 minutes).

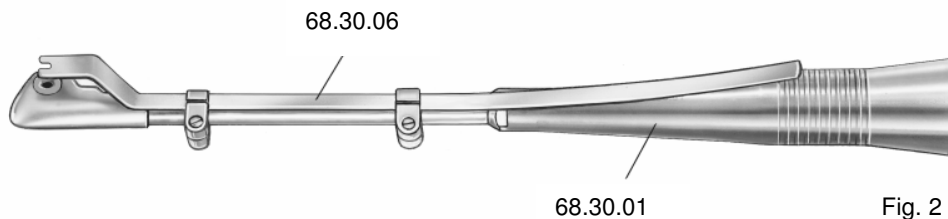
Fig. 2

The screw holding device 68.30.06 is attached to the shank of the angular screwing instrument 68.30.01, and fits all screw inserts (screw inserts in MEDICON brochure: Servotronic EC100).

Mounting of the fixation clamps:

1. Apply the upper and lower part of the fixation clamp to the shank of the angular screwing instrument, push against each other and grip as indicated in Figs. 3 and 4.

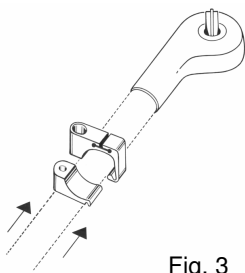
Fig. 3

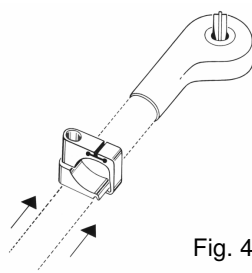
Fig. 4


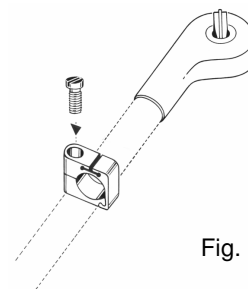
Fig. 5

2. Insert the fixing screw (Fig. 5) and screw in. Wait before tightening the screw. The fixation clamp must still permit sufficient movement for alignment.
3. Apply the second fixation clamp using the same method described in point 1 and 2.

Fig. 6
 Illustration of a fixation clamp with the fixing screw (enlarged Illustr.).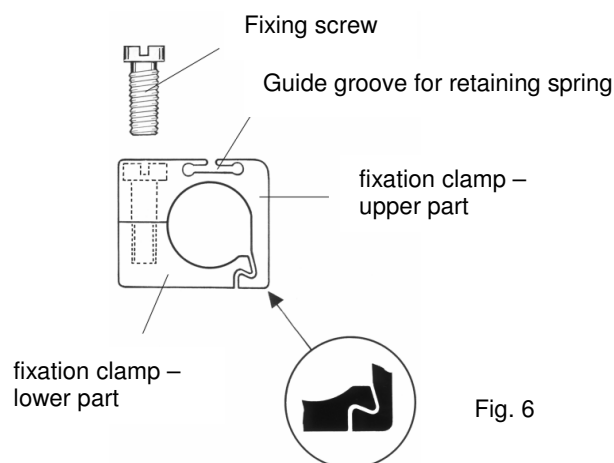

Apply the retaining spring:

1. Push the two fixation clamps against each other (Fig. 7). Insert the retaining spring in accordance with the guide grooves on both fixation clamps and push through (Fig. 8).

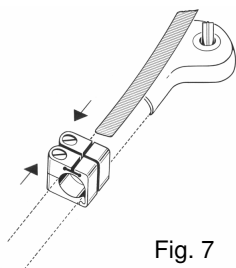
Fig. 7

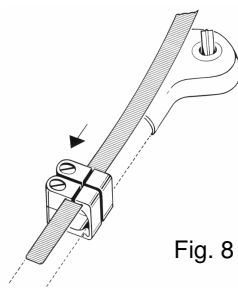
Fig. 8

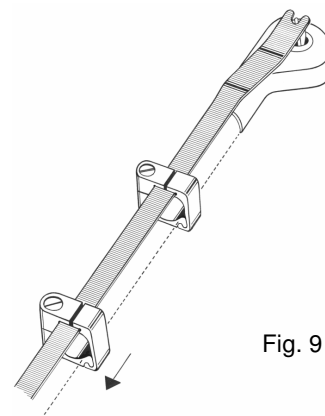
Fig. 9

2. Distance the two fixation clamps from each other (Fig. 9) and fix at approximately the distances indicated in the general illustration (Fig. 2). Align the screw holding device and tighten the fastening screws of the fixation clamps. Please ensure that the fixation clamps are aligned parallel to the retaining spring.

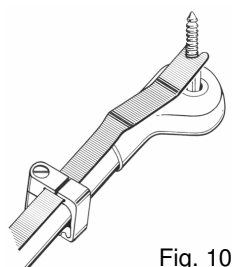
Fig. 10

Tip: As a visual aid when aligning the screw holding device, it is advisable to insert a screw insert in the angular screwdriver. This allows the correct position of the screw holding device to be set before tightening the fastening screws. In addition, using a suitable implant screw, it is possible to check whether the screw holding device is functioning correctly (Fig. 10).

Note:

The screw holding device may only be used in conjunction with the angular screwdriver and for the intended purpose only.

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