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Connecting Cable for Radiofrequency Surgery
(monopolar, bipolar)

Contents

- Application
- Connection to RF unit
- Cleaning, disinfection, Sterilization
- Repairs
- Storage

Application

RF cables are used to connect monopolar and bipolar electrodes and instruments to RF units. When correctly used, these products are subject to wear and tear to a greater or lesser extent depending on intensity of use. This wear and tear is a technical limitation and unavoidable. For this reason, the product must be given a thorough visual and functional inspection before each application. If the product reveals outwardly recognizable deficiencies or does not perform in the manner described in these instructions, contact the manufacturer or his authorized representatives immediately.

The products must be used only in combination with MEDICON accessories and only by clinically trained personnel.

Caution:

The cables must be fully inspected for visible abnormalities. Cables with brittle or defective insulation must not be used.

Connection to RF unit

- Before first use, confirm that the cable is suitable for the unit used by you.
- Connect only those instruments to the cable whose connectors fit the cable exactly.
- Prevent mechanical damage to the cable and do not kink the cable. Defective cables can fundamentally not be repaired – they must in any event be exchanged for new ones.
- To prevent monitor image distortion, RF cables must not be installed directly in parallel with camera cables. RF cables must never lie directly in contact with the patient's skin. Do not lay the cable in loops.
- Comply with the operating instructions of the RF unit as well as its general information on electrosurgical interventions.
- When pulling the RF cables, always grip the connector. Never pull on the RF cable itself, since this could cause damage.

Caution:

Cables with defective/brittle insulation or a breakage of the electrical conductor can cause burns to the user/patient or a fire. RF cables in direct contact with the skin can result in capacitive currents causing burns.

Cleaning, disinfection, sterilization

Caution:

The RF cables must not be cleaned in an ultrasonic bath. The RF cables must not be sterilized in hot air sterilizers.

For disinfection, cleaning and sterilization, remove the connecting cable from the instruments. Immediately after use, wipe over the cable with a

surface disinfectant containing alcohol. Alternatively, the RF cable can also be placed in a combined cleaning solution and disinfectant, but this can reduce its service life through oxidation of the contacts or crystallizing out in the connectors. Please study the information from the chemical manufacturer. Afterwards, rinse the cable thoroughly with sterilized water. Only cleaned and disinfected instruments must be sterilized.

For sterilisation use only the following sterilisation procedures:

- Fractionated vacuum procedure with sufficient product drying
- Steam sterilizer validated according to EN 13060 respectively DIN EN 13485 and DIN EN 554.
- Sterilisation time and temperature:
At least 5 minutes residence time at 134°C

Repair

Basically, RF cable must not be repaired. They must be exchanged for new ones.

Storage

Temperature: -20°C - +50°C (-4°F - +122°F)
Relative atmospheric humidity: 0 - 75%, noncondensing
Atmospheric pressure: 500 -1600 mbar

The products must be stored with protection against direct sunlight. Before first use, storage in the original packaging is recommended.

MEDICON eG, as supplier of these products, is not liable for defects caused by or arising from improper use or handling, especially through non-observance of the enclosed operating instructions or through improper cleaning or maintenance.

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