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Suction and Irrigation cannulae

Contents

- Application
- Cleaning
- Sterilization
- Storage

Application

The instruments are used for suction and irrigation in minimal invasive surgery.

Cleaning

Preparations for decontamination

Reprocess the instruments through manual or mechanical processing as soon as possible after each application. Separable instruments must be disassembled because otherwise areas that could have been in contact, even indirect contact with the patient may not be cleaned, which could pose a risk of infection. For manual processing, system-specific brushes and cleaning guns are required. The instruments should also be treated by ultrasound cleaning if available.

Instruments must be deposited ready for rinsing on trays that are appropriate for machine washing. The instrument trays must be constructed in such a way, that subsequent cleaning in an ultrasound or cleaning and disinfecting device won't be hampered by acoustical or cleaning shadows. Thorough rinsing of the instruments must be ensured by connecting them to the washing load carrier, attaching them to special nozzles, etc. Unconnected inner lumens are not reached by the processing medium and are therefore not cleaned and disinfected.

Please do only use suitable pH-neutral or alkaline cleaning agents. The choice of the cleaning agent depends on the properties of the instruments and on the national guidelines and recommendations.

In case of elevated chloride concentration of the water pitting or stress corrosion may occur. By using fully desalinated water or alkaline cleaning agents the danger of such corruptions will be minimized.

Thermal disinfection

Use fully desalinated water. The thermal disinfection process takes place at 80-95°C and leaving it to work correspondingly according to the A0-concept, EN ISO 15883-1.

Drying

Please ensure a sufficient drying phase with the cleaning and disinfecting device.

Cleaning agents

Please observe all parameters off the manufacturer concerning concentration, temperature and reacting time. Automatic doser must be controllable.

Maintenance, control and checking

After cleaning / disinfection the instruments must be macroscopically clean, i.e. free of visible residues. Visual checking. Instruments that are not sufficiently cleaned must be cleaned once again and rinsed sufficiently afterwards.

Prior to assembly the valve pistons and miniature taps must be sprayed

with spray oil (Medicon Item No. 46.00.40). Exchange defective instruments (fissures, deformation or abrasion) since they do not fulfil their function any more or not safely enough.

Attention:
Handle the valve pistons with utmost care. Damaging the surface leads to ruining the instrument.

Sterilization

Never sterilize instruments before cleaning.

For sterilization do only apply the following sterilization procedures:






- Fractionated vacuum procedure with sufficient product drying
- Steam sterilizer validated according to EN 13060 respectively DIN EN 13485 and DIN EN 554.
- Sterilization time and temperature:
At least 5 minutes hold time at 134°C

The user is responsible for validating the recommended parameters, in order to make allowances for possible differences with respect to sterilization chambers, packing methods and loading configurations. The same applies to usage not complying with the recommendations. It is indispensable to achieve a SAL (Sterility Assurance Level) of 10^{-6} .

Storage

Store instruments in a dust-free and dry place. To prevent condensate formation, major temperature fluctuations should be avoided. Do not store chemicals together with instruments. The storage life depends on the type of packaging.

MEDICON eG, as supplier of these products, is not liable for defects caused by or arising from improper use or handling, especially through non-observance of the enclosed operating instructions or through improper cleaning or maintenance.

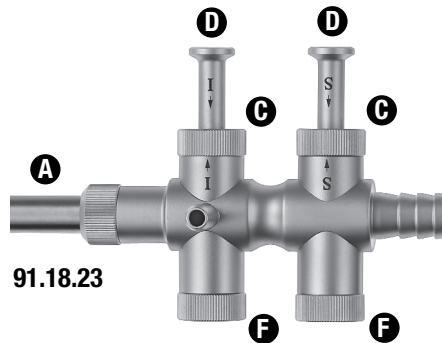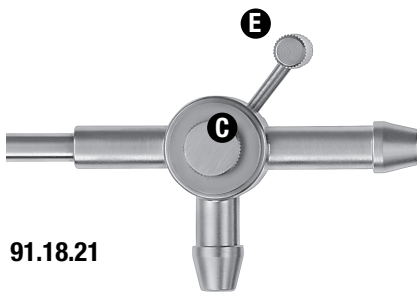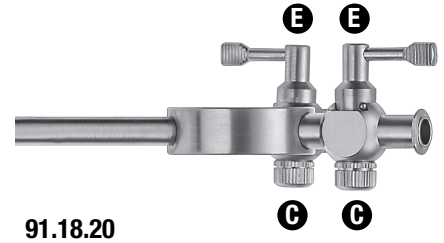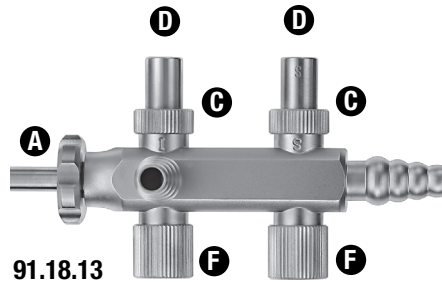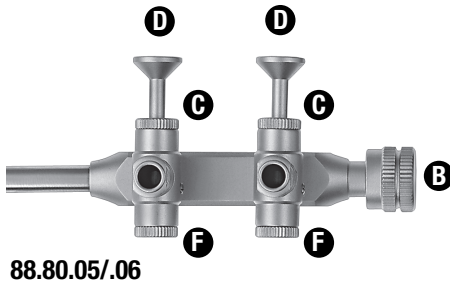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Assembly

Prior to assembly spray the valve pistons and miniature taps with spray oil (Medicon Item No. 46.00.40).
The assembly is done in the reverse order as that described above under Disassembly.

Attention: When inserting the valve pistons, please take care to ensure that the letters "I"(Irrigation) and "S"(Suction) on the housing and the valve pistons are above one another. Finally, check the trumpet valves and taps for ease of movement.