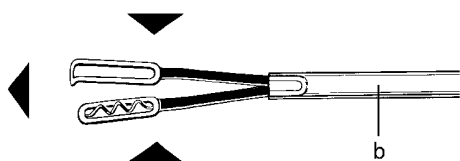
Montage

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2. Insert the electrode shaft b through the tubular shaft c up to the stop of the handle e while pushing button a.
3. Release button a and rotate the electrode shaft b till button a engages noticeable.

Preparation, Sterilization:

Please use for cleaning of the tubular shaft c and the shaft seat of handle e a soft cleaning brush f, for example MEDICON 909.10.45.

Please use for removing residues of coagulation from the grasping parts only soft cleaning brushes. Please don't use metal brushes.

Before operating with bipolar instruments they should be checked individually for eventual defects in insulation and proper function. Please also refer to the operating instructions for minimally invasive surgery, non-steril, reusable.

We recommend cleaning and disinfection in dismantled condition. Before first use, the brandnew instruments need to be lead to a manual or mechanical preparation.

Recommended method of sterilization: Steam sterilization (134°C, 2 bar, induction time at least 5 minutes, or 121°C, 1 bar, induction time at least 15 minutes). **Hot air sterilization is prohibited.**

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Bipolar forceps, dismantable

Intended use:

Instrument for bipolar coagulation of tissue/vessel.

Legend:

- a = button
- b = electrode shaft
- c = tubular shaft
- d = knurled nut
- e = handle
- f = cleaning brush

Dismantling:

1. Press button a and hold it depressed.
2. Hold the electrode shaft b on top of the shaft and pull it off the tubular shaft c.
3. Screw off the tubular shaft c from the handle e at knurled nut d in direction of arrow L.

Assembling:

1. Screw the tubular shaft c into the handle e at knurled nut d in direction of arrow R.

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