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Dismountable Bipolar Coagulation Forceps

Table of contents

- Application
- Assembly and operation
- Cleaning
- Sterilization
- Storage

Application

The MEDICON dismountable bipolar coagulation forceps was developed for application in minimally invasive surgery, especially for laparoscopy, and serves for tissue manipulation, grasping and cutting. The instrument is inserted through a 5-mm trocar sleeve.

The coagulation is carried out with electrical energy from electrosurgical HF generators. This instrument can be connected at the bipolar output of Erbe, Martin, Berchtold, Codman or Valleylab HF generators, or of other comparable devices.

The output power in bipolar mode should not exceed 60 W when using this instrument.

Bipolar coagulation instruments may be used only by persons that have received special training or instruction for doing so.

Service and repairs should be carried out by appropriately trained and qualified persons only. Please contact MEDICON or your medical technology department if you have any questions in this respect.

The dismountable bipolar forceps was developed for sterilization in an autoclave, which is a proven sterilization method for this instrument.

Assembly and operation

Follow the enclosed pictograms for assembling or disassembling the instrument.

When the instrument has been correctly assembled and connected via a suitable bipolar cable, the instrument can be held either in the right hand or in the left hand.

To close the instrument jaws: close the handle.

To open the instrument jaws: open the handle.

The jaw parts are not moved axially by opening or closing the handle, meaning they are not pulled into the instrument tube.

Cleaning

Preparations for decontamination

Reprocess the instruments through manual or mechanical processing as soon as possible after each application. Separable instruments must be disassembled because otherwise areas that could have been in contact, even indirect contact with the patient may not be cleaned, which could pose a risk of infection. For manual processing, system-specific brushes and cleaning guns are required. The instruments should also be treated by ultrasound cleaning if available.

Instruments must be deposited ready for rinsing on trays that are appropriate for machine washing. The instrument trays must be constructed in such a way, that subsequent cleaning in an ultrasound or cleaning and disinfecting device won't be hampered by acoustical or cleaning shadows. Thorough rinsing of the instruments must be ensured by connecting them to the washing load carrier, attaching them to special nozzles, etc. Unconnected inner lumens are not reached by the processing medium and are therefore not cleaned and disinfected.

The connecting cable must be removed prior to cleaning and disinfection of monopolar and bipolar instruments. Please do only use suitable pH-neutral or alkaline cleaning agents. The choice of the cleaning agent depends on the properties of the instruments and on the national guidelines and recommendations.

In case of elevated chloride concentration of the water pitting or stress corrosion may occur. By using fully desalinated water or alkaline cleaning agents the danger of such corruptions will be minimized.

Thermal disinfection

Use fully desalinated water. The thermal disinfection process takes place at 80-95°C and leaving it to work correspondingly according to the AO-concept, EN ISO 15883-1.

Drying

Please ensure a sufficient drying phase with the cleaning and disinfecting device.

Cleaning agents

Please observe all parameters off the manufacturer concerning concentration, temperature and reacting time. Automatic doser must be controllable.

Maintenance, control and checking

After cleaning / disinfection the instruments must be macroscopically clean, i.e. free of visible residues. Visual checking.

Critical areas like articulations or serrated jaws must be checked thoroughly. Instruments that are not sufficiently cleaned must be cleaned once again and rinsed sufficiently afterwards. Instruments with flexible parts must be cooled and maintained with oil before the function test. Instruments with ratchets like clamps and needle holders may only be closed until the first detent (danger of stress cracking). Exchange defective instruments (fissures, deformation or abrasion) since they do not fulfil their function any more or not safely enough.

Prior to each use of the monopolar and bipolar instruments, the instruments must be checked again, individually, for possible damage to the insulation.

Sterilization

Never sterilize instruments before cleaning.

For sterilization do only apply the following sterilization procedures:

- Fractionated vacuum procedure with sufficient product drying
- Steam sterilizer validated according to EN 13060 respectively DIN EN 13485 and DIN EN 554.

- Sterilization time and temperature:
At least 5 minutes hold time at 134°C

Storage

Store instruments in a dust-free and dry place. To prevent condensate formation, major temperature fluctuations should be avoided. Do not store chemicals together with instruments. The storage life depends on the type of packaging.

MEDICON eG, as supplier of these products, is not liable for defects caused by or arising from improper use or handling, especially through non-observance of the enclosed operating instructions or through im-proper cleaning or maintenance.

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