Reduzierer

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Reduction Pieces

Contents

1. Application
2. Cleaning
3. Sterilisation
4. Storage
5. Spare parts

1. Application

The instruments are used to adapt trocar sleeves in minimally invasive abdominal surgery.

Adapter sleeves (91.10.40, 91.10.41 and 91.10.42) are inserted entirely into the trocar sleeve.

Slide adapters (91.16.00, 91.16.01) are clicked onto the trocar sleeve casing and the sealing cap fixture can be pivoted if required.

Adapter with lug (91.16.10, 91.16.11, 91.16.12, 91.16.13). The lug is first put over the trocar casing and the adapter can then be inserted into the trocar sleeve as required.

2. Cleaning

Preparations for decontamination

Reprocess the instruments through manual or mechanical processing as soon as possible after each application. Separable instruments must be disassembled because otherwise areas that could have been in contact, even indirect contact with the patient may not be cleaned, which could pose a risk of infection. For manual processing, system-specific brushes and cleaning guns are required. The instruments should also be treated by ultrasound cleaning if available.

Instruments must be deposited ready for rinsing on trays that are appropriate for machine washing. The instrument trays must be constructed in such a way, that subsequent cleaning in an ultrasound or cleaning and disinfecting device won't be hampered by acoustical or cleaning shadows. Thorough rinsing of the instruments must be ensured by connecting them to the washing load carrier, attaching them to special nozzles, etc. Unconnected inner lumens are not reached by the processing medium and are therefore not cleaned and disinfected. Please do only use suitable pH-neutral or alkaline cleaning agents. The choice of the cleaning agent depends on the properties of the instruments and on the national guidelines and recommendations. In case of elevated chloride concentration of the water pitting or stress corrosion may occur. By using fully desalinated water or alkaline cleaning agents the danger of such corrosions will be minimized.

Thermal disinfection

Use fully desalinated water. The thermal disinfection process takes place at 80-95°C and leaving it to work correspondingly according to the A0-concept, EN ISO 15883-1.

Drying

Please ensure a sufficient drying phase with the cleaning and disinfecting device.

Cleaning agents

Please observe all parameters off the manufacturer concerning concentration, temperature and reacting time. Automatic doser must be controllable.

Maintenance, control and checking

After cleaning / disinfection the instruments must be macroscopically clean, i.e. free of visible residues. Visual checking.

Instruments that are not sufficiently cleaned must be cleaned once again and rinsed sufficiently afterwards. Exchange defective instruments (fissures, deformation or abrasion) since they do not fulfil their function any more or not safely enough.

All plastic parts must be checked after sterilisation to make sure that they are not cracked, brittle or worn. If necessary, these parts must be replaced by new genuine parts (see section Spare Parts).

3. Sterilization

Never sterilise instruments before cleaning.

For sterilisation do only apply the following sterili-sation procedures:

- Fractionated vacuum procedure with sufficient product drying
- Steam sterilizer validated according to EN 13060 respectively DIN EN 13485 and DIN EN 554.
- Sterilisation time and temperature:
At least 5 minutes hold time at 134°C

4. Storage

Store instruments in a dust-free and dry place. To prevent condensate formation, major temperature fluctuations should be avoided. Do not store chemicals together with instruments. The storage life depends on the type of packaging.

5. Spare parts

Reduction pieces	Sealing cap
91.10.40	909.11.40
91.10.41	909.11.43
91.10.42	909.11.40
91.16.00	909.11.40
91.16.01	909.11.43
91.16.10	909.11.40
91.16.11	909.11.40
91.16.12	909.11.43
91.16.13	909.11.40

MEDICON eG, as supplier of these products, is not liable for defects caused by or arising from improper use or handling, especially through non-observance of the enclosed operating instructions or through improper cleaning or maintenance.